



Kem Medical Products Corp.

KemSafe™ Solution Neutralizer for Glutaraldehyde and OPA disinfectant solutions

(Glycine based formulation)

Catalog # 9074-1G

Directions:

1. Pour **KemSafe™** Solution Neutralizer into container with waste disinfectant solution, which you intend to neutralize (one 16 oz. bottle neutralizer per 1 gallon waste solution).
2. Reaction is complete in 5 minutes (a color change will occur), the neutralized solution is then safe for disposal.

Note: If the **KemSafe™** Solution Neutralizer is to be used in a scope processor, the processor must be purged after neutralization has occurred. It is recommended that a water rinse follow the evacuation of the neutralized disinfectant solution.

Always maintain adequate ventilation in accordance with applicable regulations.

Disposal is subject to federal, state, and municipal sewer regulations.

Precaution:

This product is not to be ingested or brought into contact with the skin and eyes. Review manufacturer's Safety Data Sheet. Keep away from children.

Rev 2

February 1, 2011