

**Cat. No.  
9500- (P/pail  
or T/tote)  
One Kit**

# KemSure

## UNIVERSAL SPILL CONTROL KIT

For Use on spills up to 1/2 gallon of liquid

Contents:

- One (1) Pair Nitrile Gloves
- One (1) AAMI Level-3 Isolation Gown
- One (1) Procedure Mask with Elastic Ear loops
- One (1) Full Length Face Shield
- One (1) Pair Non-Skid Knee-High Boot Covers
- One (1) Lightweight Bouffant Cap
- Two (2) Spill Response Pillows (Each absorbs approx. ¼ gallon)\*\*\*  
\*\*\*Additional Pillows (set of 2) may be purchased with Catalog #9500-RP\*\*\*
- One (1) Temporary Disposal Bag with Label

### DIRECTIONS:

1. Open your **KemSure**® Universal Spill Control Kit.
2. Remove contents from kit. Put on gown, booties, gloves, mask, cap, and face shield as necessary.
3. Place the Chemsorb™ Spill Pillow on spill area and allow it to absorb the liquid.
4. Absorption is complete in about 5 minutes.
5. Place used pillows in temporary spill bags and seal (**each pillow will absorb 1/4 gallon of liquid**).
6. Contents should be labeled and management notified. Dispose of material in accordance with Federal, State, and local regulations.
7. Replace Universal Spill Kit -- Re-Order -- Cat. No. **9500- (P/pail or T/tote)**. To purchase individual components of this kit, please see the Inventory Card included inside.

### **DO NOT REUSE PILLOWS**

The **KemSure**® Universal Spill Control Kit is designed for small spills, up to 1/2 gallon. Each Pillow will pick- up approximately 32 oz. of liquid. Spills should be cleaned up by trained personnel only. Dispose of spilled materials in accordance with Federal, State and local regulations. Avoid generating dust. In case of eye contact, flush with large quantities of water. Read Safety Data Sheet (SDS) for information on product safety and handling procedures.

Chemsorb™ is a Trademark of  
Precision Laboratories, Inc.

**Caution:**  
**Do not use Chemsorb™ Pillows  
for Hydrofluoric acid.  
Keep out of Reach of Children.**