

Cat. No.  
9540- (P/pail  
or T/tote)  
One Kit

**KemSure**<sup>®</sup>

## OPA-C NEUTRALIZING SPILL CONTROL KIT

### For Use on OPA-C Disinfectant Spills

#### Contents:

- One (1) Pair Nitrile Gloves
- One (1) AAMI Level-3 Isolation Gown
- One (1) Procedure Mask with Elastic Ear loops
- One (1) Full Length Face Shield
- One (1) Pair Non-Skid Knee-High Boot Covers
- One (1) Lightweight Bouffant Cap
- KemSure<sup>®</sup> Neutralizing powder for up to 5 Liters of solution
- Two (2) Spill Response Pillows (Each absorbs approx. ¼ gallon)<sup>\*\*\*</sup>  
**\*\*\*Additional Pillows (set of 2) may be purchased with Catalog #9500-RP\*\*\***
- One (1) Temporary Disposal Bag with Label

#### DIRECTIONS:

1. Open your **KemSure**<sup>®</sup> Neutralizing Spill Kit.
2. Remove contents from kit. Put on gown, booties, gloves, mask, cap, and face shield as necessary.
3. Sprinkle **KemSure**<sup>®</sup> Neutralizer Powder evenly over entire spill area. The powder is designed to use entire contents of jar in one application (**Neutralizes 5 liters of liquid**).
4. Neutralization is complete in 5 minutes.
5. Place the Chemsorb<sup>™</sup> Spill Pillow on spill area and allow it to absorb the neutralized solution.
6. Place used pillows in temporary spill bags and seal (**each pillow will absorb 1/4 gallon of liquid**).
7. Contents should be labeled and management notified. Dispose of material in an appropriate and responsible manner.
8. Replace Neutralizing Spill Kit, -- Re-Order -- Cat. No. 9540- (P/pail or T/tote).
9. Replace Neutralizing Powder Container -- Re-Order -- Cat. No. 9540-R.

#### **DO NOT REUSE PILLOWS**

The **KemSure**<sup>®</sup> Neutralizing Spill Control Kit is designed for OPA-C disinfectant spills. Each Pillow will pick up approximately ¼ gallon of neutralized liquid (1/2 gallon total), while the OPA-C Neutralizing Powder will neutralize up to 5 liters of an OPA-C spill. Use enough pillows to absorb the entire volume of the spill. Spills should be cleaned up by trained personnel only. Powder must remain sealed in closed container until use. Dispose of spilled materials in accordance with Federal, State and local regulations. Avoid generating dust. In case of eye contact, flush with large quantities of water. Read Safety Data Sheet (SDS) for information on product safety and handling procedures.

Chemsorb<sup>™</sup> is a Trademark of  
Precision Laboratories, Inc.

**Caution:**  
Do not use Chemsorb<sup>™</sup> Pillows  
for Hydrofluoric acid.  
Keep out of Reach of Children.