

**Cat. No.**  
**9540-S**  
**One Kit**

# KemSure

## OPA-C Residual/Surplus Neutralizing Kit

For Safe Disposal of Residual/Surplus OPA-C Solution

Contents:

- **Eight (8) Containers of OPA-C Residual/Surplus Neutralizing Powder**

*Each container neutralizes the residual/surplus OPA-C left in a used container of OPA-C HLD solution (up to 2.28 oz)*

### DIRECTIONS:

1. Open your **KemSure**® OPA-C Residual/Surplus Neutralizing Kit.
2. Remove one container of **KemSure**® OPA-C Residual/Surplus Neutralizing Powder and open it.
3. Open the used container of OPA-C HLD solution and pour the **KemSure**® OPA-C Residual/Surplus Neutralizing Powder into it.
4. Neutralization is complete in less than 5 minutes.
5. Dispose in accordance of local State and Federal Regulations.
6. Replace **KemSure**® OPA-C Residual/Surplus Neutralizing Kit, -- Re-Order -- Cat. No. **9540-S**.

The KemSure® OPA-C Residual/Surplus Neutralizing Kit is designed to neutralize the residual/surplus OPA-C solution left in a used container. Dispose of spilled materials in accordance with Federal, State and local regulations. Read Safety Data Sheet (SDS) for information on product safety and handling procedures.

**Caution:**  
**Keep out of Reach of Children.**