

**Cat. No.**  
**9560- (P/pail  
or T/tote)**  
**One Kit**

# KemSure

## OPA/GLUTARALDEHYDE NEUTRALIZING SPILL CONTROL KIT

For Use on OPA/Glutaraldehyde Spills

Contents:

- One (1) Pair Nitrile Gloves
- One (1) AAMI Level-3 Isolation Gown
- One (1) Procedure Mask with Elastic Ear loops
- One (1) Full Length Face Shield
- One (1) Pair Non-Skid Knee-High Boot Covers
- One (1) Lightweight Bouffant Cap
- KemSure® Neutralizing powder for two (2) gallons of solution
- Two (2) Spill Response Pillows (Each absorbs approx. ¼ gallon)\*\*\*
- \*\*\*Additional Pillows (set of 2) may be purchased with Catalog #9500-RP\*\*\*
- One (1) Temporary Disposal Bag with Label

### DIRECTIONS:

1. Open your **KemSure®** OPA/Glutaraldehyde Neutralizing Spill Kit.
2. Remove contents from kit. Put on gown, booties, gloves, mask, cap, and face shield as necessary.
3. Sprinkle **KemSure®** OPA/Glutaraldehyde Neutralizer Powder evenly over entire spill area. The powder is designed to use entire contents of jar in one application (**Neutralizes 2-gallons of liquid**).
4. Neutralization is complete in 5 minutes.
5. Place the Chemsorb™ Spill Pillow on spill area and allow it to absorb the neutralized solution.
6. Place used pillows in temporary spill bags and seal (**each pillow will absorb 1/4 gallon of liquid**).
7. Contents should be labeled and management notified. Dispose of material in accordance with Federal, State, and local regulations.
8. Replace OPA/Glutaraldehyde Neutralizing Spill Kit, -- Re-Order -- Cat. No. **9560- (P/pail or T/tote)**.
9. Replace OPA/Glutaraldehyde Neutralizing Powder Container, -- Re-Order -- Cat. No. **9560-R**. To purchase individual components of this kit, please see the Inventory Card included inside.

### **DO NOT REUSE PILLOWS**

The **KemSure®** Neutralizing Spill Control Kit is designed for OPA-C disinfectant spills. Each Pillow will pick-up approximately ¼ gallon of neutralized liquid, while the OPA/Glutaraldehyde Neutralizing Powder will neutralize up to 2 gallons of solution. Spills should be cleaned up by trained personnel only. Powder must remain sealed in closed container until use. Dispose of spilled materials in accordance with Federal, State and local regulations. Avoid generating dust. In case of eye contact, flush with large quantities of water. Read Safety Data Sheet (SDS) for information on product safety and handling procedures.

Chemsorb™ is a Trademark of  
Precision Laboratories, Inc.

**Caution:**  
**Do not use Chemsorb™ Pillows  
for Hydrofluoric acid.**  
**Keep out of Reach of Children.**