

## **KemSafe™ Powder Neutralizer for Glutaraldehyde and OPA disinfectant solutions**

*(Glycine based formulation)*

**Catalog # 9075-G5 / 9075-G5X**

### **Directions:**

1. Pour **KemSafe™** Neutralizing Powder into container with waste disinfectant solution that you intend to neutralize (one container neutralizing powder per 5 gallons waste solution). Entire container contents are to be used in a single use.
2. Stir solution to mix.
3. Reaction is complete in 5 minutes (a color change will occur); the neutralized solution is then safe for disposal.

**Note:** If the KemSafe™ Neutralizer Powder is to be used in a scope processor, the processor must be purged after neutralization has occurred. It is recommended that a water rinse follow the evacuation of the waste solution.

**Note:** Always maintain adequate ventilation in accordance with applicable regulations. Disposal is subject to federal, state, and municipal sewer regulations.

### **Precaution:**

This product is not to be ingested or brought into contact with the skin and eyes. Review manufacturer's Safety Data Sheet. Keep away from children.

The Glutaraldehyde/OPA powder Neutralizer contains no hazardous or toxic chemicals, therefore Kem Medical recommends it is safe for disposal. It is always prudent to check with local, state, and Federal environmental regulations.