

KemSafe™ Powder Neutralizer for Glutaraldehyde and OPA disinfectant solutions

(Glycine based formulation)

Catalog # 9075-G8 / 9075-G8X

Directions:

1. Pour **KemSafe™** Neutralizing Powder into container with waste disinfectant solution that you intend to neutralize (one container neutralizing powder per 8 gallons waste solution). Entire container contents are to be used in a single use.
2. Stir solution to mix.
3. Reaction is complete in 5 minutes (a color change will occur); the neutralized solution is then safe for disposal.

Note: If the KemSafe™ Neutralizer Powder is to be used in a scope processor, the processor must be purged after neutralization has occurred. It is recommended that a water rinse follow the evacuation of the waste solution.

Note: Always maintain adequate ventilation in accordance with applicable regulations. Disposal is subject to federal, state, and municipal sewer regulations.

Precaution:

This product is not to be ingested or brought into contact with the skin and eyes. Review manufacturer's Safety Data Sheet. Keep away from children.

Rev 3
September 23, 2016