

# KemSafe™ Solution Neutralizer for Glutaraldehyde and OPA disinfectant solutions

*(Glycine based formulation)*

Catalog # 9074

## Directions:

1. Pour **KemSafe™** Solution Neutralizer into container with waste disinfectant solution, which you intend to neutralize (one 32 oz. bottle neutralizer per 2 gallons waste solution or 8 parts waste solution).
2. Reaction is complete in 5 minutes (a color change will occur), the neutralized solution is then safe for disposal.

*Note:* If the **KemSafe™** Solution Neutralizer is to be used in a scope processor, the processor must be purged after neutralization has occurred. It is recommended that a water rinse follow the evacuation of the neutralized disinfectant solution.

Always maintain adequate ventilation in accordance with applicable regulations. Disposal is subject to federal, state, and municipal sewer regulations.

## Precaution:

This product is not to be ingested or brought into contact with the skin and eyes. Review manufacturer's Safety Data Sheet. Keep away from children.

Rev 3  
September 23, 2016