

## **KemSafe™ Powder Neutralizer for Glutaraldehyde and OPA disinfectant solutions**

*(Glycine based formulation)*

Catalog # 9075-Q

### **Directions:**

1. Pour **KemSafe™** Neutralizing Powder into container with waste disinfectant solution that you intend to neutralize (one container neutralizing powder per 1 quart waste solution). Entire container contents are to be used in a single use.
2. Stir solution to mix.
3. Reaction is complete in 5 minutes (a color change will occur); the neutralized solution is then safe for disposal.

**Note:** If the KemSafe™ Neutralizer Powder is to be used in a scope processor, the processor must be purged after neutralization has occurred. It is recommended that a water rinse follow the evacuation of the waste solution.

**Note:** Always maintain adequate ventilation in accordance with applicable regulations. Disposal is subject to federal, state, and municipal sewer regulations.

### **Precaution:**

This product is not to be ingested or brought into contact with the skin and eyes. Review manufacturer's Safety Data Sheet. Keep away from children.

Rev 3  
September 23, 2016