

KemSafe™ Solution Neutralizer for Glutaraldehyde and OPA disinfectant solutions

(Glycine based formulation)

Catalog # 9074

Directions:

1. Pour **KemSafe™** Solution Neutralizer into container with waste disinfectant solution, which you intend to neutralize (one 32 oz. bottle neutralizer per 2 gallons waste solution or 8 parts waste solution).
2. Reaction is complete in 5 minutes (a color change will occur), the neutralized solution is then safe for disposal.

Note: If the **KemSafe™** Solution Neutralizer is to be used in a scope processor, the processor must be purged after neutralization has occurred. It is recommended that a water rinse follow the evacuation of the neutralized disinfectant solution.

Always maintain adequate ventilation in accordance with applicable regulations. Disposal is subject to federal, state, and municipal sewer regulations.

Precaution:

This product is not to be ingested or brought into contact with the skin and eyes. Review manufacturer's Safety Data Sheet. Keep away from children.

The Kem Medical KemSure 9074 Glutaraldehyde/OPA solution neutralizer does not contain any hazardous or toxic components. Therefore, it is safe for disposal down the drain.