

## Instructions for Use

### **KemSure® Premium Sterilization Barcode Labels**

1. Load the roll of labels according to the printer manufacturer's instructions.
2. Print desired barcode per user software or interface.
3. Peel the printed label from the backing.
4. Place on desired location.
5. Using 2 fingers, press firmly from the middle and move your fingers outward to the edges of the label.

KemSure Premium Sterilization Barcode Labels have been validated 100% sterilization efficacy for Steam, EtO and Hydrogen Peroxide by independent laboratory testing. Separate validation reports for Steam, EtO and Hydrogen Peroxide are available upon request.

Revision 2.1 – October 4, 2019